

# Pureline<sup>®</sup>

*By Supera Anesthesia Innovations*

## Operators Manual OC6000 & OC6200



**OC6000**



**\*OC6200**

**\* Anesthesia machine NOT included**

**SUPERA** | ANESTHESIA  
INNOVATIONS

*Copyright 2013*

## **Examination and Preparation for Use**

We are delighted to have you as a Supera Anesthesia Innovations customer and want you to be completely satisfied with your purchase. Please inspect the contents of your order to see if everything is as you expected. Should anything not be exactly right or if anything was damaged in shipping, please contact your sales representative right away for help.

Our goal is to make your new machine as easy to use and care for as possible.

This device is meant to be operated under the normal surveillance and control of a veterinarian trained in its use. However, you need to know more about this device than just how to operate it.

**Please read this manual in its entirety before using the machine.**

If you have any comments or questions, we welcome the opportunity to address them.

Please contact us directly at 877-620-1500.

Thank you!

Brian Lawson  
President,  
Supera Anesthesia Innovations

**Proudly Designed and  
Made In Oregon, USA**

# Machine Assembly

**\* NOTE: THIS REQUIRES TWO PEOPLE TO SAFELY ASSEMBLY \***



1. Carefully unpack the top assembly and frame from the box

2. Align the top to the frame as shown.

NOTE: The frame base has two cross bars with holes in them for mounting the concentrator. That is the front of the frame.

*\*See next page for fastening to frame*



3. Assemble the top assembly with the frame by installing the 1/4-20 X 1 3/4" long screws and washers provided.

4. Tighten the screws with the 3/16" allen wrench provided VERY tightly.



1. Remove the oxygen concentrator from the box and guide the 4 threaded post into the frame



2. Install and tighten the nuts.

**DO NOT OVER TIGHTEN THE NUTS, SNUG FIT IS ALL THAT IS NEEDED**

# Matrx VMS<sup>®</sup> installation

*\*Anesthesia Machine NOT included \**



Slide the post from the Matrx VMS<sup>®</sup> machine into the black adapter until it stops. Tighten the handle or set screw with the wrench provided.

Pureline<sup>®</sup> is a registered trademark owned by Lawson Enterprises, Inc. MATRX and VMS are registered trademarks owned by Midmark Corporation. All other trademarks are the property of their respective owners. Supera Anesthesia Innovations and the Pureline<sup>®</sup> OC6200 are not affiliated with nor endorsed by Midmark Corporation.

# Oxygen Setup - OC6000



Attach the green hose from your anesthesia machine to the DISS fitting.

Attach the green hose from the OC6000 top to the threaded fitting.

**\* DO NOT OVER TIGHTEN\***

# Oxygen Setup - OC6200

*\*Anesthesia Machine NOT included \**



Attach the green hose to your anesthesia machine.

# Accessories

Optional Organizer Bar  
P/N OC6220



Optional waste gas interface valve  
P/N EVC630

Optional F-Air evacuation kit  
P/N EVC555





# MAINTENANCE

Your oxygen concentrator has filters that must be kept clean for proper operation.

**P/N 8400-1025 external cabinet filter - List Price \$9.95**

Coarse filter on the back of the machine. This can be washed with mild soap and water. DO NOT INSTALL UNTIL 100% DRY

**P/N 8400-1180 Fine inlet filter (square black plastic filter) - List Price \$14.95**

The fine particle filter goes under the coarse filter. This is a replacement only filter.

**Failure to monitor the condition of the filters will void the warranty.**

**Contact Supera for replacement  
877-620-1500**

# Warranty / Return Policy

Supera Anesthesia Innovations is proud to offer our customers the best warranty in the veterinary industry—a full three years.

## **Supera LLC Limited Warranty**

Supera LLC guarantees its products to be free of defects in design, materials, and workmanship for Three years from the date of purchase.

The Supera LLC Limited Warranty covers parts and construction of all products manufactured by Supera LLC in its USA manufacturing facility. If we find that a Supera machine does not operate as stated in the instructions and specifications that come with the product due to defects in design, materials, or workmanship, Supera will repair the unit or replace it with one of equal or greater value at no charge to the customer for up to three years from the date of purchase.

## Specific Exclusions from this Warranty

The Supera Limited Warranty does not cover—and Supera LLC shall not be liable for—the following: (1) repairs and replacements required because of misuse, abuse, negligence, alteration, accident, freight damage, or tampering; (2) products that are not installed, used, and properly cleaned as required in the Supera LLC “Installation” and/or “Installation/Operation” manual applicable for the product; (3) products considered to be of a consumable nature such as rubber or plastic goods (4) accessories or parts not manufactured by Supera Anesthesia Innovations; (5) specially manufactured products; (6) damage caused by animals; (7) charges by anyone (including Supera Anesthesia Innovations authorized dealers) for adjustments, repairs, replacement parts, installation, or other work performed upon or in connection with such products which is not expressly authorized in writing in advance by Supera LLC; (8) products manufactured by other companies and resold by Supera LLC. This includes medical gas products. Any warranties on these items are controlled directly by the manufacturer of these items to the original purchaser. Information on these manufacturer’s warranties will be enclosed with the applicable products. In addition, Supera LLC will furnish copies of any of the warranties controlled by any such manufacturers upon request.

## For Warranty Service

Please contact Supera Customer Service at 877-620-1500 (toll-free in the USA) or +1 503 723 5068 (for local and International calls) or email us at [info@superavet.com](mailto:info@superavet.com).

One of our customer service representatives will be able to answer your questions and provide information on having your machine serviced under warranty by an authorized Supera technician or by returning the unit to Supera for warranty service.

# DOCUMENTATION

Date purchased:

Purchased from:

Machine serial number:

Machine service information

Service Date:

Service information:

Service Date:

Service information:

Service Date:

Service information:

Pureline® is a registered trademark owned by Lawson Enterprises, Inc. MATRX and VMS are registered trademarks owned by Midmark Corporation. All other trademarks are the property of their respective owners. Supera Anesthesia Innovations and the Pureline® OC6200 are not affiliated with nor endorsed by Midmark Corporation.